



Roots Vet Dental Supplies Ltd

Tel: 01455 698449 LowCall Tel: 0844 8700109 Fax: 0844 8700108

Email: info@rootsvetdental.co.uk www.rootsvetdental.co.uk

LUBRICATION OF DENTAL HANDPIECES

Airmotor

(Speed adjustable 4 - 25,000 rpm)




Air turbine highspeed handpiece

(Speed 300-400,000 rpm)





These handpieces can have either mid-west or borden connections.

Use the pointed nozzle with your aerosol oil to lubricate in these holes.

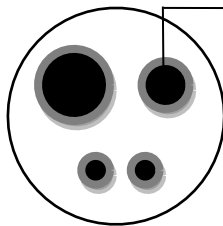

Midwest

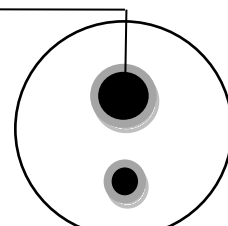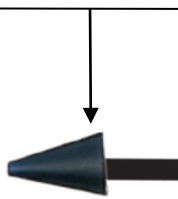

Borden


Straight Handpiece E Fitting



Contra-Angle slow speed Handpiece E Fitting



Use the E-fitting nozzle with your aerosol oil to lubricate in this hole.

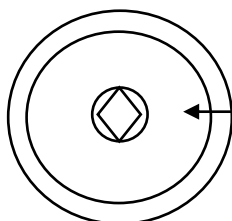

E-fitting (base of handpiece)



We recommend once all dentals are complete and pre and post autoclave process that you lubricate all handpieces and store them in a cup/pot with the handpieces upside down to allow the oil to pass through all the working parts and to clear any debris that may have accumulated.

**FOR MAINTENANCE, ADVICE OR DENTAL SUPPLIES PLEASE DON'T HESITATE TO CONTACT US.
0844 8700109**